

Easy Wi-Fi Set-Up

1. Press Settings

- From the MedMinder home-screen select the translucent grey icon on the top-right.



2. Enter Pin Code

- Enter your custom pin code, or the default numeric pin of 1111.

3. Click “Open Wi-Fi Settings”

4. Select Your Wi-Fi

- From the list of available wireless networks, select your Wi-Fi network name.
- If your network is not displayed on the list, click “REFRESH” on the top-right.

5. Enter Wi-Fi Password

- Click the “Enter Password” field.
- Type in your Wi-Fi password using the on-screen keyboard.
- Click “CONNECT” on the bottom-right.

6. Wait For Connection

- Once connected successfully, the word “Connected”, and your Wi-Fi network name will change from black to blue.

7. Return Home

- Click the back icon on the top-right twice to arrive back at the home-screen.



Congratulations on setting up Wi-Fi for your MedMinder!