

Inserting the MedMinder Pharmacy Tray



1 Open the MedMinder lid by pulling up on the front of the unit.



2 Remove everything from the device, such as an empty tray or cups.



3 Open the box and remove the full tray of medications.



4 Place the tray into the MedMinder with the triangular notch facing you.



5 Gently peel the plastic cover off of the tray.



6 Close the lid after ensuring that the cups are securely in place.

* Use the included key to lock the MedMinder on the front of the unit, if needed.

Cómo insertar la bandeja de farmacia MedMinder



1
Abra la tapa del MedMinder tirando la parte frontal hacia arriba de la unidad.



2
Retire todo del dispositivo, como la bandeja o recipientes vacíos.



3
Abra la caja y retire la bandeja completa de medicamentos.



4
Coloque la bandeja en el MedMinder con la muesca triangular señalando hacia usted.



5
Retire con cuidado la cubierta plástica de la bandeja.



6
Cierre la tapa después de asegurarse de que los recipientes estén bien puestos.

* Utilice la llave incluida para bloquear el MedMinder en la parte frontal de la unidad de ser necesario.



MedMinder

Para más información, visite www.medminder.com o llámenos al us at 888-633-6463